Languages:

Page:

de	Gebrauchsanweisung Glass Bottle	2
en	Instructions for use Glass Bottle	3
fr	Mode d'emploi Glass Bottle	4
it	Istruzioni per l'uso Glass Bottle	5
nl	Gebruiksaanwijzing Glass Bottle	6
es	Indicaciones Glass Bottle	7
pt	Indicações Glass Bottle	8
tr	Kullanım kılavuzu Glass Bottle	9
da	Brugsanvisning Glass Bottle	10
sv	Bruksanvisning Glass Bottle	11
no	Bruksanvisning Glass Bottle	12
fi	Käyttöohjeet Glass Bottle	13
pl	Instrukcja Glass Bottle	14
gr	Οδηγίες χρήσης Glass Bottle	15

GlassBottle

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GlassBottle

For use in clinics and hospitals

for the collection and storage of breast milk.

How to use these bottles

Expressing breast milk using a breast pump.

Connect the bottle to an original ARDO PumpSet (see separate Instruction leaflet).

Storage of breast milk

Seal the bottle with the lid provided.

Breast milk can be stored as follows:

At room temperature: 6 hours at max. 25 °C; in the fridge: 5 days at max. 4 °C; in the fridge-freezer compartment, with door: 3 – 6 months; in the deep freeze: 6 – 12 months at -20 °C

Cleaning

1. Cleaning in the clinic or the hospital: sterilise at 121°C / 20 Min.
2. At home: before using for the first time - and once daily when in use – boil the bottles, with lids and seals (disassembled) for 10 minutes. The bottles are dishwasher and steam steriliser-safe. After each use, wash the bottles with dishwasher liquid and rinse thoroughly. Dry with a clean cloth and cover. Neither the bottles, nor any other parts, are microwave safe.

Check / Safety regulations

Before each use and after each cleaning, check bottles and caps for damage (glass splinters, cracks etc.) and cleanliness. Replace any damaged parts immediately. Use only after thorough sterilisation.

Material

Glass; Polypropylene

Disposal

All parts must be disposed of in accordance with local regulations.

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Ardo medical AG