en

INSTRUCTIONS FOR USE

bottle set Bottles for breast milk

Package contains:
3 bottles (150 ml), 3 lid rings, 3 lid liners

Intended use:

This breast milk bottle is used for expressing (in combination with a pumpset), storing and feeding breast milk. It is intended for multiple use and can be safely sterilized.

Expressing breast milk

To express breast milk, connect the milk bottle directly to the pumpset (e.g. Ardo pumpset).

Storage of breast milk

After expressing the milk, close the milk bottle using the lid.

Breast milk can be stored as follows:

- at room temperature 16–29 °C (60–85 °F): 3-4 hours optimally; 6-8 hours are acceptable under very clean conditions
- in the refrigerator ≤ 4 °C (39 °F): 72 hours optimally; 5-8 days are acceptable under very clean conditions
- in the freezer< −17 °C (0 °F): 6 months optimally; 1 year is acceptable

DO NOT fill more than 150 ml into bottles, as milk expands when frozen.

Feeding breast milk

Breastfeeding must be well established before using other feeding methods.

Thaw the breast milk, which was stored in the freezer, first in the refrigerator or at room temperature. Then warm the breast milk to about 37 °C (98 °F) by immersing the bottle in warm water or holding under lukewarm running water.

Breast milk should not be heated in a microwave.

Manual cleaning

Before using for the first time and after each time it is used:

- Dismantle the product into its individual parts
- Rinse for at least 1 minute using cold tap water.
- Clean the components of the bottle with commercial detergent and warm water (not over +40°C) and rinse with cold tap water. Make sure that all detergent residues are removed.
- A soft brush can also be used for heavy soiling.
- Allow all parts of a clean surface to air dry.

Manual disinfection

1x daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.

Check the individual parts for cracks and changes to the surface according to the cleaning and disinfection process.

If you discover any changes, dispose of the part affected and replace with a new one.

Material

Milk bottle, lid ring, lid liner: Polypropylene

Important notice: Plastic bottles and lids may become brittle if frozen and damaged if dropped. DO NOT use breast milk if bottles and lids are damaged.

Disposal

According to the local regulations

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