ardo melia

Instructions for use | Gebrauchsanweisung | Mode d'emploi |
Istruzioni per l'uso | Instrucciones de uso | Gebruiksaanwijzing

Instruction
Video

 ardo
SWITZERLAND

Instruction for Use

en	Ardo Melia – Wearable double electric breastpump	3–10
de	Ardo Melia – Tragbare elektrische Doppel-Milchpumpe	11–18
fr	Ardo Melia – Tire-lait électrique double pompage portable	19–26
it	Ardo Melia – Tiralatte elettrico portatile doppio	27–34
es	Ardo Melia – Sacaleches eléctrico doble portátil	35–42
nl	Ardo Melia – Draagbare elektrische dubbelzijdige borstkolf	43–50

Thank you for choosing Ardo.

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
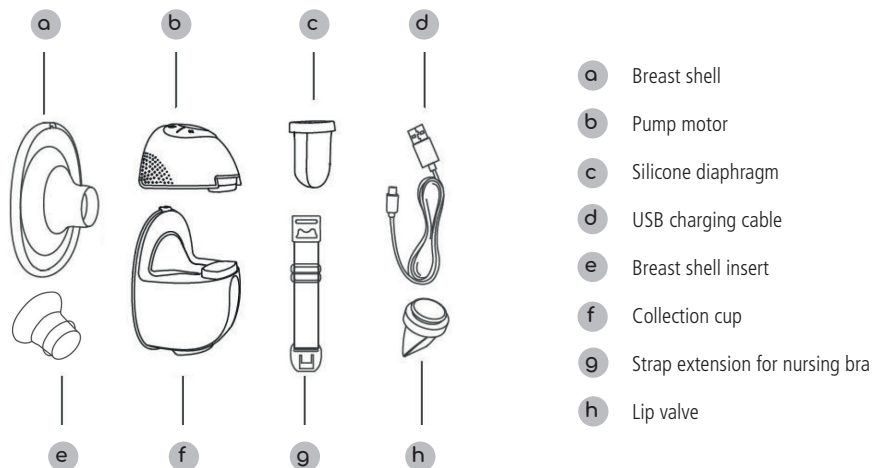
1. Product description

Breast shells and inserts in various sizes as well as spare parts are available, including from online shops (see also section 13).

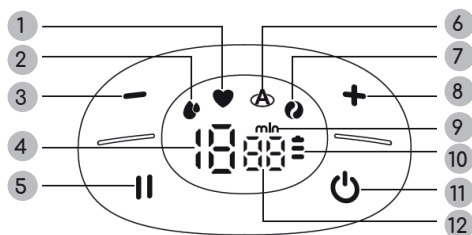2. Display and button functions

1	Stimulation mode	7	Massage mode
2	Expression mode	8	Increase vacuum
3	Decrease vacuum	9	Abbreviation for "minutes"
4	Vacuum level	10	Rechargeable battery display
5	Start/Pause button	11	On/Off and Mode selection button
6	Automatic mode	12	Expressing time in minutes

Please note

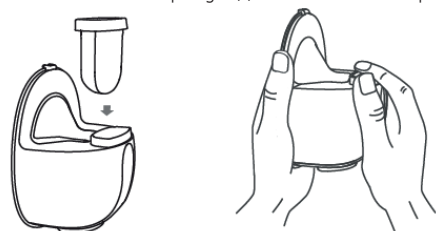
- Only use in cleaned condition (see also section 8). Check for damage after cleaning.
- For optimal use of the Ardo Melia, we recommend regularly replacing the silicone parts (breast shells, breast shell inserts, diaphragm and lip valves). These parts should be replaced at the first sign of visible damage, or if the breastpump ceases to function properly.
- Do not use during pregnancy because expressing can trigger contractions.
- Wash hands thoroughly with soap before use.

3. Assembly


1. Fit the lip valve (h) onto the collection cup (f) as shown. Ensure the correct orientation of the lip valve.


2. Insert the silicone diaphragm (c) into the collection cup (f) as shown and press down firmly.

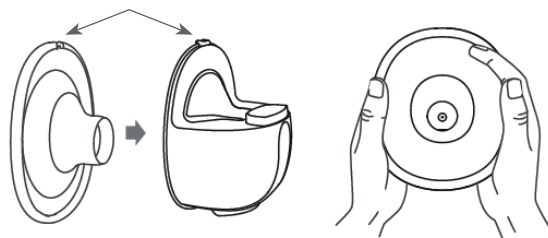3. Insert the breast shell (a) into the collection cup (f). The notches (marked with arrows below) must be aligned. Using both hands, pull the entire rim of the breast shell (a) over the edge of the collection cup (f) and press firmly into place.

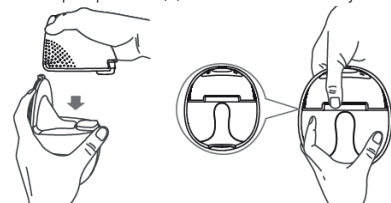
4. Fit the pump motor (b) as shown. Press firmly into place.

4. Expressing

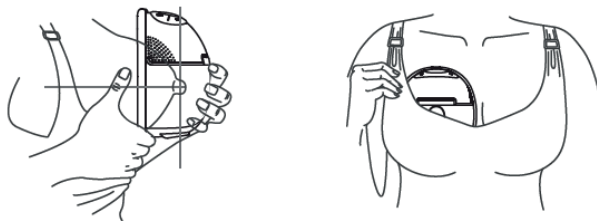
1. Place the pump on the breast so that the nipple is positioned in the middle of the funnel. (If necessary, adjust the size of the funnel using a breast shell insert (e) or a different size of breast shell (a), see also section 9). Pull the bra cup up over the pump to hold the pump in place on the breast.

2. Hold down the  button for approximately 3 seconds until the display lights up. (The lettering colour of the display is light, in contrast to the image on the front page. A darker colour was chosen for the image for better visibility.)

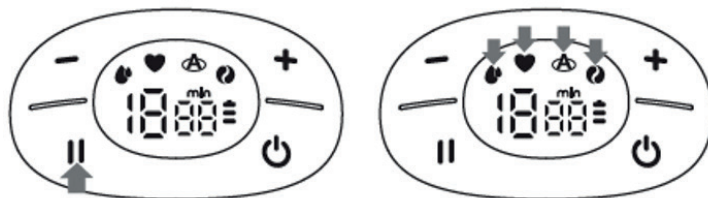
3. Press the  button to "Start" and "Pause". Select a mode or switch between modes using the  button.

- **Stimulation mode**  : Triggers the let-down reflex and starts the flow of breast milk. Can be set at the beginning of a pumping session until the breast milk starts to flow. By pressing the mode selection button you can then switch to expression mode, for example.
- **Expression mode**  : Used for efficient expressing of breast milk after triggering the let-down reflex.
- **Automatic mode**  : starts with the lowest vacuum level and then moves up one level every 15 seconds up to level 12 (out of a total of 15 possible levels). If you press the "+" or "-" buttons during this process, the Ardo Melia switches to manual mode and stays at the corresponding vacuum level. You can adjust the vacuum level by pressing the "+" or "-" buttons.
- **Massage mode**  : shorter, quicker cycle to stimulate milk flow.

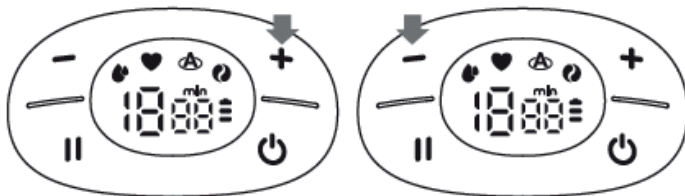4. Each mode has 15 vacuum levels. You can increase the vacuum by pressing the "+" button; you can decrease it by pressing the "-" button.

5. To turn off the Ardo Melia, press and hold the  button until the display goes out.

Tips:

- The Ardo Melia has an in-built, automatic memory function: when you turn on and start the pump, it starts pumping in the mode and at the vacuum level that you used last time, before you turned the pump off. An exception to this is the automatic mode, which always starts at the lowest vacuum level (1).
- When the Ardo Melia is in pause mode and no buttons are pressed for 5 minutes, the breastpump automatically turns off.
- The Ardo Melia automatically turns off after 20 minutes of operation.

5. Rechargeable battery display and charging the battery

The battery symbol on the display shows the rechargeable battery charge level. If only one flashing bar is displayed, the battery is almost flat and needs to be charged. To do this, please use the supplied charging cable and a suitable plug.

Ideally, use a plug with 5 volts and 1-2 amps output power (details on plug usually 5 V, 1-2 A) as this allows you to fully charge the pump in the fastest way possible (takes about 2 hours). Plugs with a lower power rating mean the pump takes longer to charge fully.

As soon as three constantly lit bars are displayed during charging, the battery is fully charged. Now the pump has at least 100 min of expressing time. This duration may vary depending on the selected mode and vacuum level set.

6. Posture and placement in the bra

An upright or slightly reclined posture is ideal for expressing. In a very reclined position there is a risk that the expressed milk could leak out. Expressing while lying down is not possible as the expressed milk leaks out.

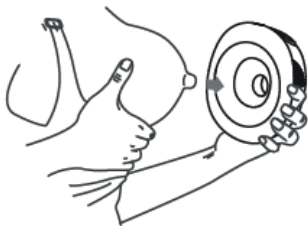
The Ardo Melia should not fit too tightly inside the bra, otherwise you risk affecting the expressing of the breast milk. If the pump is pressing too hard against your breast, loosen your bra straps. If necessary, you can also fit your nursing bra with the supplied bra strap extensions (g).


Bra tips: Choose a bra made of a stretchy material. Ardo Melia will usually fit into a breastfeeding bra better than into a normal bra. Some women who are breastfeeding find that a bustier is better than a bra.

7. Stopping expressing


1. When you have finished expressing, press and hold the On/Off button until the display goes out.
2. Take the Ardo Melia off the breast and hold it upright or stand it upright.

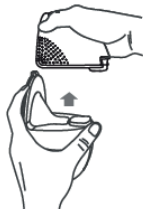3. Disconnect the pump motor (b) from the collection cup (f).

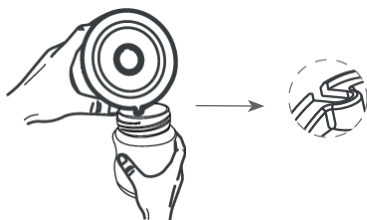4. Pour the milk through the opening in the collection cup (f, at the top) into a prepared breast milk bottle (e.g. from the Ardo Bottle Set) or a breast milk storage bag (e.g. Ardo Easy Store).

8. Cleaning and sterilisation

Cleaning: before using for the first time and after each use

- Dismantle the Ardo Melia into its individual parts
- Rinse all parts (except for the pump motor, b) in cold tap water for 1 minute
- Then clean using a mild dish soap and warm water (not over +40 °C)
- Rinse using lots of clean water.

Warnings:

- o Take care when removing and cleaning the lip valve (h), as the Ardo Melia will no longer work correctly if it is damaged.
- o To clean the lip valve (h), please rub it gently in warm water with a couple of drops of dish soap.
Do not put any objects into the lip valve, as this may lead to damage. Do not clean the lip valve with a washing-up brush or similar utensils.
- Alternatively, you can clean the parts in their dismantled state (apart from the pump motor, b) in the dishwasher.
- Drying: Place all the parts on a clean, unused tea towel or paper towel and leave them to air dry thoroughly in a clean place.

Sterilisation: 1x daily after cleaning

- Boil all the parts (apart from the pump motor, b) in their dismantled state for 5 minutes.
- Ensure that there is enough water in the saucepan (the parts must be fully submerged in water); deformations can occur if there is not enough water.
- Alternatively, you can sterilise the parts in an Ardo Easy Clean bag in the microwave.
- After sterilisation, leave the parts on a clean surface to air dry thoroughly.

Check the individual parts for cracks and changes to the surface after each cleaning and sterilisation process. If you notice changes, replace the defective part.

Pump motor: Do not rinse or submerge the pump motor (b) in water. If needed, wipe with a clean cloth moistened with 70% alcohol (ethanol or isopropanol).

If you are in a hard water area, limescale deposits can build up on pump parts after several cleaning and disinfection cycles, causing parts to look cloudy. This can be prevented or removed by adding a few drops of lemon juice to the water when disinfecting parts.

9. The right size of breast shell

A suitable breast shell (a) is needed to express successfully. If the breast shell is too big or too small, this may cause sore nipples and may be painful. You should therefore use the right breast shell size for you or an appropriate insert (e).

The size is right if:

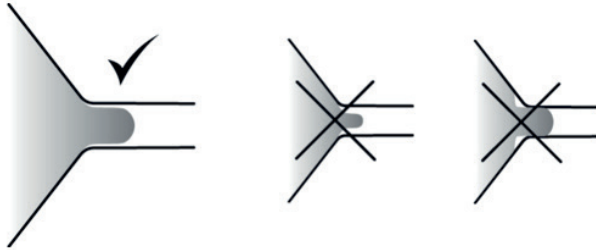
- in the funnel there is around 2 mm space free around the nipple.
- the nipple moves freely in the funnel during expressing and can follow the rhythmic movements of the pump.
- there is no areolar tissue in the funnel.

- the milk flows and the breast feels soft all over after expressing.

On the flaps of the Ardo Melia retail box there are templates you can use to measure the diameter of your nipples.

Cut off the flaps and measure the diameter of your nipples. This makes it easy to find the correct size of breast shell (a) or breast shell insert (e) where needed.

If your nipple rubs uncomfortably against the inner side of the funnel, use a larger breast shell (a). If the areola is also sucked into the funnel, use a smaller breast shell insert (e).

Two breast shells (a) in size 24 mm and two breast shell inserts (e) in size 19 mm are included in delivery. Breast shells in size 27 mm and breast shell inserts in size 13, 15, 17, 19 and 21 mm are also available for the Ardo Melia (see also section 13), subject to change.

10. Storing breast milk

Recommended storage time at room temperature: Up to 4 hours at +25°C. In the refrigerator: up to 4 days at +4°C. In a freezer compartment with a door: 6 months up to 1 year at -18°C.

Do not heat breast milk in the microwave.

11. Possible problems and troubleshooting

Weak or no vacuum:

- Ensure that the Ardo Melia is correctly assembled, in particular check whether the rim of the breast shell (a) is fully pulled over the edge of the collection cup (f) (press firmly into place with both hands). Also check whether the lip valve (h) is correctly aligned.
- Ensure that the individual parts are not damaged, particularly the lip valve (h).

Pain when expressing:

- Ensure that the nipple is in the middle of the funnel.
- Check that the size of the breast shell (a) / breast shell insert (e) is right for your nipples.
- Check that the vacuum intensity is not set too high.
- Contact a professional for help.

12. Warranty

There are legally prescribed minimum warranty periods from the date of purchase that are specific for each country. The warranty coverage is limited to the first purchaser.

12.1. General terms and conditions

Ardo medical AG guarantees that its products are free of material and manufacturing defects. The warranty period starts from the date of purchase. Defective materials are replaced during the warranty period free of charge, provided that the warranty has not been voided by improper use or access to the device. To ensure the warranty coverage and flawless function of the device, the instructions for use must be followed and only Ardo accessories used. The warranty is void if the device has been opened or manipulated.

13. Additional accessories and spare parts

- 63.00.472 Breast shell insert for Hands-Free / Ardo Melia 13 mm (2 pcs)
- 63.00.473 Breast shell insert for Hands-Free / Ardo Melia 15 mm (2 pcs)
- 63.00.455 Breast shell insert for Hands-Free / Ardo Melia 17 mm (2 pcs)
- 63.00.439 Breast shell insert for Hands-Free / Ardo Melia 19 mm (2 pcs)
- 63.00.471 Breast shell insert for Hands-Free / Ardo Melia 21 mm (2 pcs)

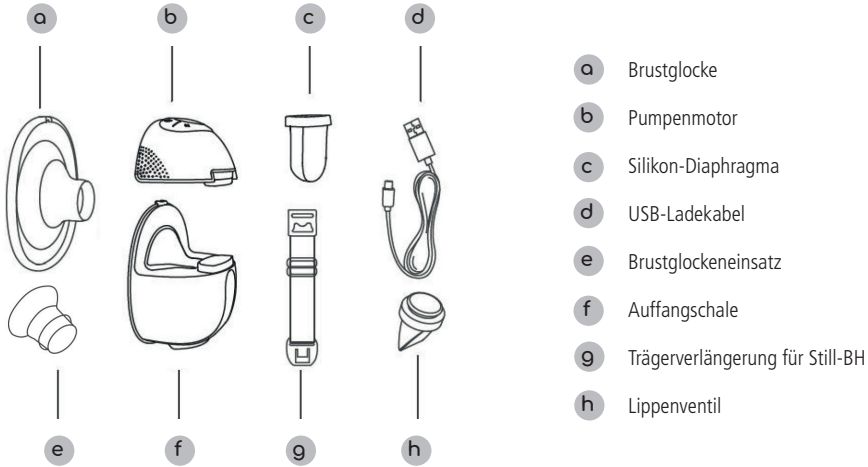
Please note that breast shell inserts from traditional Ardo pumpsets are not compatible with the Ardo Melia. The breast shell inserts from the Ardo Hands-Free pumpset are compatible with the Ardo Melia.

- 63.00.584 Ardo Melia Spare Part Set Complete (2 pcs)
- 63.00.580 Breast shell for Ardo Melia 24 mm (2 pcs)
- 63.00.581 Breast shell for Ardo Melia 27 mm (2 pcs)
- 63.00.582 Lip valve for Ardo Melia (2 pcs)
- 63.00.583 Diaphragm for Ardo Melia (2 pcs)

Subject to change without notice

Ardo offers a wide range of other products that make handling, saving and storing breast milk easy and comfortable. Ardo also provides practical breastfeeding guides and nipple care products. Ardo products are available from online shops and retailers.

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