

# ardo melia

## WEARABLE DOUBLE ELECTRIC BREASTPUMP



## Hands-free pumping – stay on the move

Ardo Melia: the new, wearable breast-pump for hands-free and discreet pumping and maximum freedom of movement.



### The advantages

- **Mobile**  
directly in the bra – without cables and tubes – for maximum of movement
- **Discreet**  
worn in the bra, compact in design, quiet in operation
- **Relaxed**  
hands-free, no need to constantly hold the pumpsets
- **Perfect fit**
  - Includes 2 breast shell/insert sizes: 19 and 24 mm
  - 13, 15, 17, 21 and 27 mm additionally available
  - Contains 2 strap extensions for the nursing bra
- **Hospital-quality pumping**  
Individually adjustable with four different modes, each with 15 vacuum steps – for maximum comfort and optimal milk flow
  - max. vacuum: 350 mbar/260 mmHG
  - max. cycles: 135/min



Mobile



Discreet




Relaxed


## Four different modes for maximum comfort and optimal milk flow

- Massage or stimulation mode gets the milk flowing


Massage: fast cycle, deep vacuum (vacuum manually adjustable)

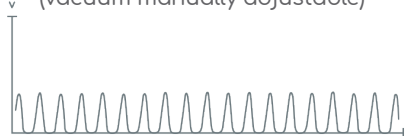

Stimulation: fast cycle, a vacuum peak every 10th time (vacuum manually adjustable)

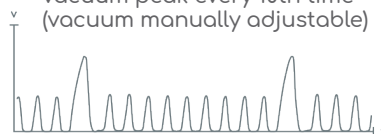


- As soon as the milk flows, switch to expression mode

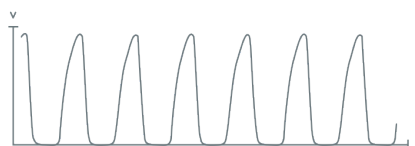

Slow cycle, high, comfortable vacuum (vacuum manually adjustable)

- Or alternatively automatic mode from the start.

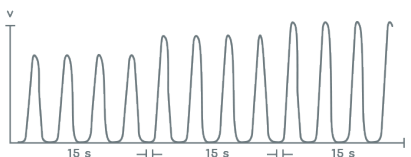

Starts at the lowest vacuum level and moves up one level every 15 seconds up to level 12 out of a total of 15 possible levels. Use the + or - button to switch to manual mode at any time and adjust the vacuum to your own requirements.

## More facts

Powerful battery (1200 mAh) for short charging time (2 h) and long expression time (min. 100 min)

With strap extensions

Silent

Soft breast shells for a snug fit and a comfortable feel

Closed system, everything stays hygienically clean

Large capacity of 2 x 180 ml / 2 x 6 fl. oz

Ardo is WHO code compliant

Free from BPA & BPS – for the baby's health

We from the Swiss family business Ardo medical guarantee the highest quality and first-class customer service.


instruction video



**ardo**  
SWITZERLAND

Ardo medical AG • [www.ardomedical.com](http://www.ardomedical.com)

## Simple and easy to use

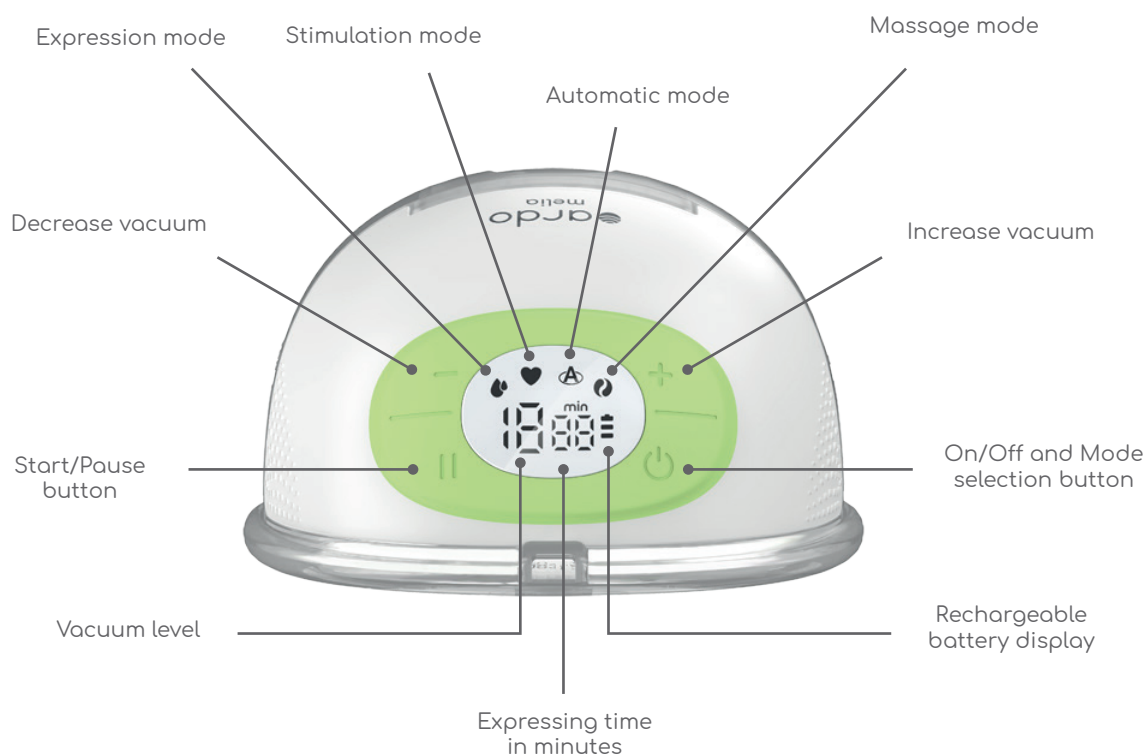

## Intelligent LED display

- Clear information about the selected mode and vacuum strength
- Enables easy adjustment and monitoring during expression
- Can also be operated in the dark
- With pause button for short interruptions
- Automatic safety switch-off after 20 minutes

## Easy handling

- The assembled breastpump can be placed upright for easy handling, as can the milk collection tray when separated from the pump motor.
- Pump motor with knobbed structure for easy grip.
- Quick and easy assembly and cleaning of the few individual parts.
- Clean in the dishwasher, sterilise in the microwave - practical and time-saving.