ardo sns

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en

Intended use

The Ardo supplemental nursing system is a special breastfeeding aid, which can be used to combine additional feeding and breastfeeding of your baby. It is mainly used in situations in which:

- the baby requires nutrition in addition to breast milk (e.g. for premature babies),
- the mother is not (yet) producing enough milk,
- the baby has difficulties with sucking or
- adoptive or foster parents would like to develop a breastfeeding relationship.

Safety instructions

- Ideally, milk should be fed at body temperature (max. 37 °C); never heat it in the microwave!
- Check the system before each use and dispose of it if damaged or soiled.
- Keep the system out of the reach of children.
- For hygiene reasons, this product is only intended for use by one person.

Preparation

- Before use, wash your hands thoroughly using soap and water.
- Fit the silicone valve with both tubes (3) onto the valve holder (4) and press firmly.
- Now feed the tubes through the attachment ring (5) from above and push the valve with holder into it.
- Pour the desired amount of milk into the container (2) and screw the attachment ring with valve onto the container.
- Clamp both tubes in the grooves of the attachment ring to block the flow of milk, before turning the container over. Pass the metal rod of the lanyard (1) through the eyelet on the container (2) and pull the lanyard tight.
- Hang the container around your neck at breast height and position the tube so that it is level with the nipple. Secure the position of the tube using the adhesive strips (6).
- If you wish to feed from just one breast, you can affix the second tube onto the container with the opening facing up. To breastfeed from both sides, attach the tubes at both breast.

Use

The level of vacuum varies depending on where the tube is placed in the baby's mouth:

the vacuum is greatest in the mouth at the upper or lower lip and least at the corner of the baby's mouth.

- When your baby begins to suck at the breast with the tube in the mouth, release the tube from the attachment ring to allow milk to flow. It is important that the baby is always actively sucking at the breast and not only on the tube.
- For babies with very weak suction, the second tube can also be released a little later. This prevents negative pressure from building up in the container and increases the flow rate of the milk.

If the container is positioned higher and the second tube is released, the flow rate can be increased. If the container is positioned lower and the second tube remains clamped, the flow rate is slowed down.

When using a supplemental nursing system and for setting the optimal flow rate, it is always advisable to seek professional advice from a lactation consultant or a medical professional.

Cleaning – before using for the first time and immediately after each use

- Disassemble the system into its individual parts and clean all parts with cold water.
- Fill the container with cold water, fully assemble the system and squeeze the container to flush water through the tube.
- Repeat these steps with warm soapy water and then with clean, cold water.
- Place all parts onto a clean cloth to allow them to air dry. Ensure the system is clean and dry before storing

Disinfection – before using for the first time and once daily

- Disinfect all parts in a pan of boiling water for 5 minutes or in the microwave (e.g. with the Ardo Easy Clean).
- Place all parts onto a clean cloth to allow them to air dry. Ensure the system is clean and dry before storing.

Disposal

In accordance with local regulations.

Complies with the requirements of Regulation (EC) No. 1935/2004 for food-safe products.

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